

Cleaning Eyeglass Frames Best Practices

1. Hand wash the frame with soap and water, removing any visible dirt.



2. Spray disinfectant and cover frame thoroughly.

3. Leave disinfectant solution on frame for 10 minutes.



4. Place frame into ultrasonic cleaner with soap and water and clean for 2-5 mins.

5. Rinse frame with tap water and let dry.



6. Pour out soap and water from ultrasonic cleaner, and clean equipment after every use.

For disinfectant and ultrasonic cleaner options, please visit www.mccrayoptical.com



McCray
Optical Supply, Inc.